

# Soft-ECC®

SFT-2000

Endocervical Curettage Device

Step By Step Instructions for Use

Ver: July 2012



***Histologics LLC***

**20409 Yorba Linda Blvd.**

**Suite 119**

**Yorba Linda, CA 92886**

Phone Toll Free and FAX: (888) 738-9757

Website; [www.histologics.com](http://www.histologics.com)

Email; [support@histologics.com](mailto:support@histologics.com)

© All Rights Reserved 2011 , Histologics LLC

Fig. 1

Place the bottom tip of the Soft-ECC in the center of your palm.

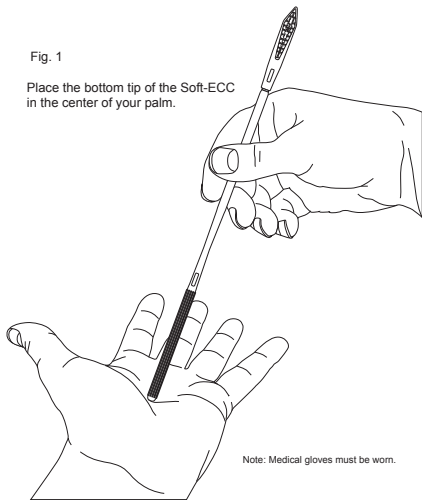


Fig. 2

Close your hand around the Soft-ECC, with your fingers along the ridges, and your forefinger on the tactile notch.


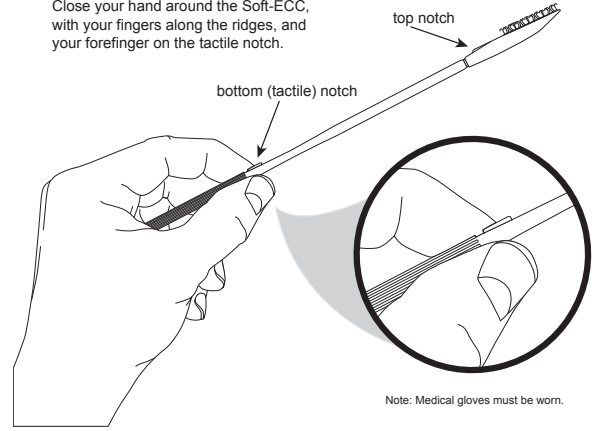


Fig. 3

Notches are aligned with Kylon™ pad. Fingers will rotate ECC device. Notches serve to keep track of rotations.

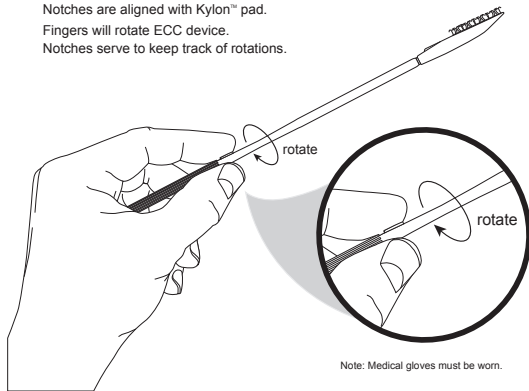

Question 4 - Fig. 4

Gently but firmly insert head of device into cervix.


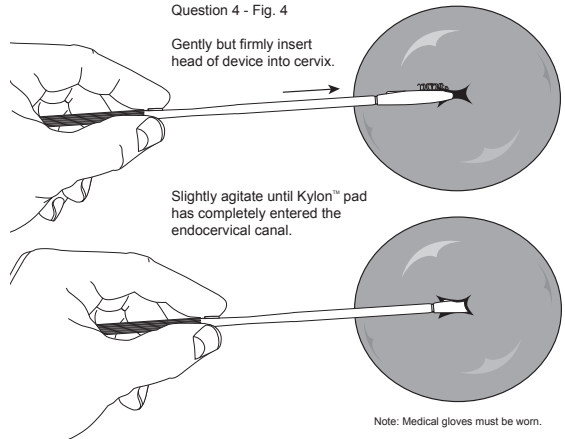


Fig. 5

Kylon™ pad on ECC head is fully positioned inside cervix.

1. Rotate clockwise 360° three complete turns.
2. Rotate counterclockwise three complete turns.

Use notches as guides.

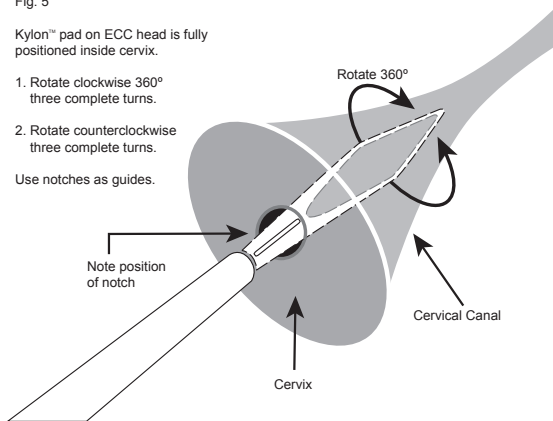

Question 4 - Fig. 6

Press the pad laterally against the canal surface. Kylon™ pad contacts cells along interior of canal. Tissue is collected by Kylon™.

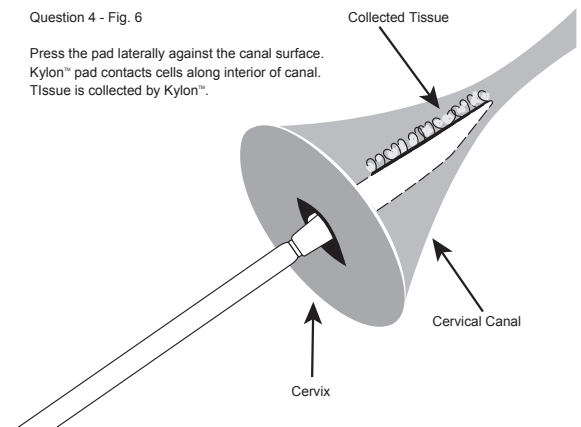


Fig. 7

After gently removing Soft-ECC® from cervix, separate head (with Kylon® pad and tissue samples) from body of device by snapping at recessed joint.

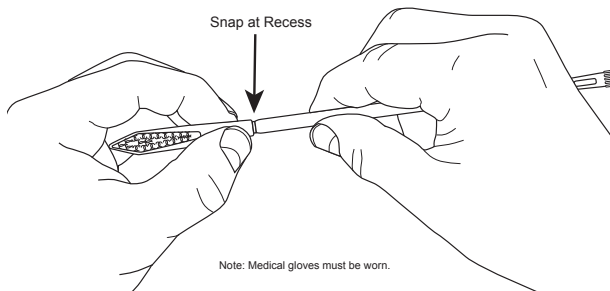


Fig. 8

Place the head of the Soft-ECC® (with Kylon® pad and tissue samples) into vial of fixative liquid and seal tightly.

